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<b>BESTDENT SELF-SEALING STERILIZATION POUCHES</b>		
<b>REFERENCES</b>	79940: 60x110 mm 79941: 70x255 mm 79942: 90x230 mm 79943: 140x250 mm	
<b>MANUFACTURER</b>	CHL MEDICAL SOLUTIONS, Srl Via Felice Cavallotti, 15 20122 Milano - Italy	
<b>CLASSIFICATION</b>	Medical Device, Class I, according to Directive 93/42/EEC	
<b>DESCRIPTION</b>	Self-sealing sterilization pouches for single use	
<b>COMPOSITION</b>	Made of medical use laminated transparent film and medical paper.	
<b>SPECIFICACIONES</b>	Sterilization with steam: the indicator colour changes from blue to black. Sterilization with ethylene oxide: the indicator colour changes from pink to yellow.	
<b>PACKAGING</b>	Box with 200 pouches	
<b>SHELF LIFE</b>	5 years from the date of manufacture	
<b>INTENDED USE</b>	Sterilization of medical devices and materials for medical use which are especially suitable for steam or ethylene oxide gas.	
<b>SAFETY INFORMATION</b>	Place the pouch with the transparent side downwards (in contact with the tray) and the paper side upwards. The objects to be wrapped must be placed in an orderly way, at the centre of the pouch, leaving about 1 cm of space between the objects and the heat sealing on the sides. The objects must be placed lengthwise with respect to the lateral heat sealing of the pouches, avoiding that any part pushes on the heat seals themselves. If these limits cannot be respected, use a larger sterilization pouch.	
<b>HANDLING AND STORAGE CONDITIONS</b>	Store in a cool dry place, away from direct sunlight. Protect from moisture. Do not use if the package is damaged.	
<b>STANDARDS APPLIED</b>	ISO 11607-1 EN 868-2,-3,-5	